




HEALQU™

The best in advanced wound care

Heel Dressing Non-adherent Foam

Waterproof, Flexible Protection

HEALQU™ Non-adherent Heel Dressing is a conformable, hydrophilic, polyurethane foam dressing with a breathable film backing. Designed to fit around the heel, this dressing has excellent fluid handling properties and is ideal for use under compression bandaging due to its low profile and low friction film backing. The non-adherent material makes dressing changes easy and as painless as possible.

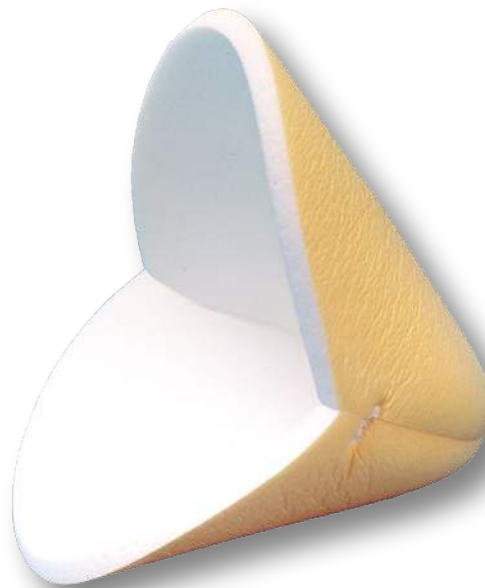

Usage

Intended for direct application to full and partial-thickness exuding wounds including:

- Pressure ulcers and diabetic ulcers
- First and second degree burns
- Cuts and abrasions
- Surgical wounds

Ref #	HCPCS	Description	QTY
9021401	A6210	~4inx5in (10cmx13cm)	5/30

For more info: 1.844.4HEALQU | www.healqu.com | info@healqu.com

INTENDED USE:

Direct application to full and partial thickness exuding wounds. This product provides excellent exudate absorption and maintains a moist, water-proof but vapor permeable healing environment.

INDICATIONS FOR USE:

Moderately to heavily exuding partial to full thickness wounds, including:

- Pressure ulcers and diabetic ulcers
- Cuts and abrasions
- First and second degree burns
- Surgical wounds

<p>ABSORBENCY</p>  <p>HIGH</p>	<p>WOUND STAGE</p> <p>1 2 3 4</p>
---	-----------------------------------

CONTRAINDICATIONS:

This product is not intended for use in the following circumstances:

- Arterial bleeds
- Heavily bleeding wounds
- Vascular fungating tumors











DIRECTIONS FOR USE:

1. Remove the heel dressing from the packaging and apply directly to heel with the skin colored side facing outward.



2. Secure the dressing with appropriate bandage or secondary dressing.

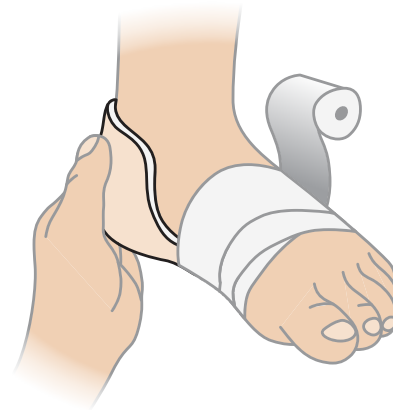

NOTES:

Change frequency should be based on the amount of exudates. Visible exudate around the edges of the dressing is a clear indication the dressing should be changed.